

Accu-Dent[®] System 2: Operatory procedure

Place in operatory:

- Water at 80-85°F (3rd bowl)
- 2 mixing bowls and spatulas
- Syringe with Pre-Sure Tip firmly attached
- 1 packet each of Syringe and Tray Gels
- Water measuring vial
- Gauze



Select
proper tray size



80-85°F water
+ Syringe
Accu-Gel



Supply of
water at
80-85°F



Mix Syringe
Accu-Gel with
water



15 – 20 Sec.

Put the
Syringe
aside

Load syringe



80-85°F water
+ Tray Accu-Gel



Supply of
water at
80-85°F



Mix
Tray Accu-Gel
and water until
smooth



30 – 40 Sec.

Put the
Impression
Tray aside

Load tray from
posterior



Syringe Gel onto
dentition, posterior
to anterior



Without Tip, syringe
Gel into vesti-
bules



Place Tray front
to back



Pour model